IB Battery 220

User's Manual

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Please read the User's Manual carefully before using the product and keep it for future reference. InBody is not liable for any accidents caused by failure to observe warnings or cautions.

Please note the important information below before reading this User Manual.

 Warning

Failure to comply with safety warnings and regulations can cause serious injury or death.

 Caution

Failure to comply with safety cautions and regulations can cause injury or property damage.

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
English

I. Product Components

- 1 IB Battery 220
- 2 Battery cable
- 3 User's Manual

1

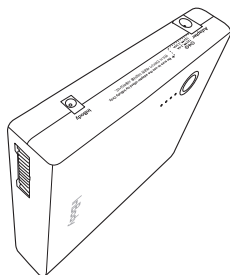IB Battery 220

2

Battery cable

3

User's Manual

Warning

- The battery is compatible with InBody's device only. Do not use the battery for any other purpose than described in the User's Manual.
- Do not place the battery near a fire or a heated source.
- Do not leave, charge or use the battery in a place where temperature may be above 60°C(140°F).
- Do not throw the battery into a fire.
- Do not short circuit the battery by touching the battery's terminals with metallic objects.
- Do not pierce the battery with a sharp object such as a screw driver.
- Do not drop or strike the battery. Such mechanical shock will cause damage to the battery and will void the manufacturer's warranty.
- If the battery is disassembled or modified beyond its original design, the manufacturer's warranty is voided.
- Do not put the battery into a microwave oven, dryer, or high-pressure container.

Caution

- Stop using the battery and contact the customer service if it becomes abnormally hot, discolored, deformed, or abnormal conditions are detected during use, charging, or storage.
- Do not use if leakage or foul odors are detected. If any liquid leaks onto your skin or clothes, wash well with water immediately.
- If liquid leaking from the battery gets into your eyes, do not rub your eyes. Wash them with water and see a doctor immediately.
- If the terminals of the battery become dirty, wipe with a dry cloth before using the battery.
- Do not submerge the product under water or keep it in a humid place for a long time. Warranty is invalidated if the battery is submerged underwater.
- Do not touch this product if your hands are wet. It may cause electric shock, fire or malfunction.
- Do not use the battery if it is flooded or submerged under the water.
- Avoid using the battery in an environment with high static electricity as it may cause a malfunction. Do not place the battery near magnets or magnetic fields as it may cause a malfunction.
- Remove the battery from the device for long-term storage and store in a place where humidity and temperature are low.
- Dispose of the battery according to local regulations.
- Abuse of this battery pack may degrade or damage the InBody device and may result in a fire.

II. How to charge the battery

The charging method of the battery can be selected using the ON/OFF button of the unit while the InBody unit is connected to the battery.


Caution

- Fully charge the battery before using it for the first time.
- Because battery lifespan is affected by the environment, it is recommended to use the battery at room temperature with low humidity.
- Make sure to only charge the battery by using the adapter provided by InBody Co., Ltd.


1 Fast charging:

- The battery may be charged while the InBody unit is powered off via exclusive adaptor included in the package. The battery's state of charge(charge level) can be confirmed by the LED light; the LED blinks while charging is in progress.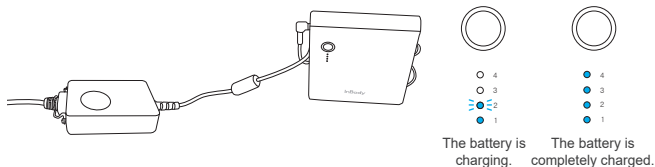


2 Slow charging:

- The battery may be charged while the InBody unit is powered on and in use via exclusive adaptor included in the package.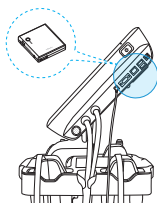

Built-in battery pack

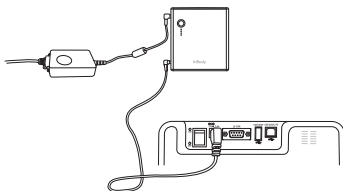External Battery Pack

Caution

- Please refer to the User's Manual for each type of connection method.


III. How to check the battery level

① Charging

The Battery level indicator(LED) blinks while the adapter is connected and charging. When all four LEDs light up and stop flashing, charging is complete.

② Checking the remaining battery level

Press and hold the button on the battery until it “beeps”. The four LEDs display the remaining battery level after about three seconds. Before the battery runs out, the last remaining LED blinks and an alarm sounds “beep-beep-” every minute. If that occurs, please charge the battery.

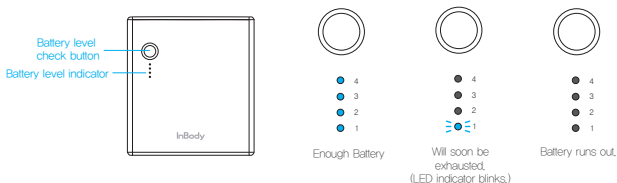③ Battery level indicator On / Off

The LEDs light up when you press the battery level check button, while charging or while connected to InBody's devices. The LEDs turn off when the battery is low.

Caution

- The battery level check button is for checking the remaining battery charge. The IB Battery 220 does not have a power on/off feature.

IV. Exterior and Functions

① Power adapter connection socket

: This socket is used to connect the adapter when the battery is in stand-alone charge mode.

② External connection socket for the BWA and InBody's device

: This socket is used to connect the BWA and InBody device and battery with the battery cable.

③ Device connection connector

: This connector is used when the battery is embedded in the BWA and InBody device.


④ Battery level indicator (LED)

: This LEDs show the remaining level of the battery.

⑤ Battery level check button

: Press and hold the button on the battery until it “beeps”.

The four LEDs display the remaining level after about three seconds.

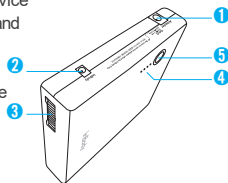IB Battery 220

V. Specifications

Model name	IB Battery 220
Rated capacity / voltage	5100mAh, 3ICR19/65-2 / 10.89V $\overline{\text{---}}$, 55.54Wh
Rated input	12V $\overline{\text{---}}$, 3.34A or 3.4A (by using the InBody's device dedicated adapter)
Charging time	Fast charging (about three hours), Slow charging (about eight hours)
Capacity display	Four-level LED capacity indication
Operation Environment	0 ~ 40°C(32°F~104°F), 30 ~75% RH, 70 ~ 106kPa
Storage Environment	-20 ~ 25°C(-4°F~77°F), 10 ~80% RH, 50 ~ 106kPa
Weight	380g (0.8 lb)

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©2022 InBody Co., Ltd. All rights reserved.
Biospace Inc. dba InBody
InBody India Pvt.Ltd.