InBody

InBodyBAND3

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Quick Start Guide **USA**

Quick Start Guide

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Snelstartgids

Guida rapida

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InBody APP

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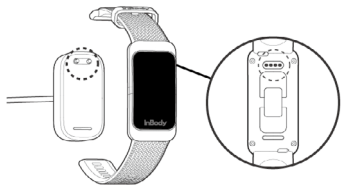
You can view and download the detailed guide on the InBody App and at <https://inbodyusa.com>

1. Power On and Charging


Connect the charger to the InBodyBAND3 charging terminal.

- * It takes approximately 1 hour and 30 minutes to fully charge the discharged battery.
- * While the battery is being charged, only the current time and remaining battery life are shown.

2. Connecting the InBody App

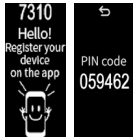- 1) Search InBody from Apple App store or Google Play store and install.
 - 2) Open the InBody App and register.
 - 3) Select InBodyBAND3 from the InBody App > ⚙ icon > Register (under Device) > InBodyBAND3
 - 4) Match the 4-digit serial number and PIN code shown at the top of the product screen with the InBody App.
- * When setup is complete, a time screen appears.

- * The InBodyBAND3 has to be registered on the InBody App to use.
- * Please update to the latest firmware version after the initial connection. The firmware can be updated from the InBody App > ⚙ icon > InBodyBAND3
- * InBody App provides InBody Test, step, sleep, heart rate, and stress results in more detail, and enables EZTraining.
- * If InBodyBAND3 is not connecting with the InBody App, try deleting InBodyBAND3 from your Bluetooth setting and reconnect it in InBody App.
- * Android OS 7.0 or later or iOS 13.0 or later.
- * Only smartphone models with Bluetooth version 5.0 or higher can be linked to InBodyBAND3.

3. Turning on the Display & Navigating

- 1) To turn on the display of the InBodyBAND3, rotate your wrist inward.
- 2) If you touch the screen, the display will turn on.
- 3) You can swipe up or down on the display to navigate.

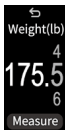
- * This product does not support the left-right scroll function.
- * If your fingers are dry, you may not be able to switch the screen smoothly.
- * You can change the type and sequence of the screen from the InBody App > ⚙ icon > InBodyBAND3 > Screen Setting > Display Setting.
- * Tap and hold the center of the screen for 3 seconds to move to the main screen.

4. InBody Test (Body Composition Analysis)

Go to the InBody Test screen and tap the center of the screen. Input your current weight. Then, tap "Measure" on the screen and cover the electrodes with your thumb and index finger.

InBody Test Screen

Weight Check / Test Start Screen

InBody Test Description Screen

InBody Test in Progress

Percent Body Fat / Skeletal Muscle Mass Height, Weight / BMI

Make sure your height is entered correctly in the  icon (System) > Profile prior to testing.

Posture for Accurate InBody test

Put your fingers on the electrodes to touch with as much skin contact as possible.

Lightly press on the electrodes with two fingers. Make sure the InBodyBAND3 is stable on your wrist.

Be careful not to let the hands touch each other.

Both upper arms should not touch the body.

* You can view the test results on your InBody App.

5. Caution Before Measurement

Warning

- A person wearing an implanted medical device (e.g., pacemaker, etc.) or a life-sustaining device (e.g., patient monitoring device, etc.) should not use the InBodyBAND3. For example, the pacemaker may malfunction due to the current flowing through the body during InBody Test and this could pose a serious risk to your health and safety. InBody Co., Ltd. will not be liable for any injury or damage from violating these warnings.
- Bioelectrical Impedance Analysis (BIA) method uses micro-current and does no harm to the human body. However, if a pregnant woman uses InBodyBAND3, her fetus may be affected. Therefore, medical advice is required for such usage.
- Do not use the InBodyBAND3 in situations that are distracting and lead to dangerous circumstances or while driving a car.
- Since InBodyBAND3 is not a medical device, it is not intended to diagnose or give medical direction based on the symptoms of a disease.

Caution

- InBody Test results may be affected by environmental factors and the customer's measurement condition.
- InBodyBAND3 has a built-in magnet. Keep a safe distance from objects that can be affected by magnets, such as credit cards and medical devices.
- InBodyBAND3 was tested in a controlled environment and will be waterproof or dustproof only under the specified circumstance (IP68 requirements of IEC 60529 satisfied). Submerging the product underwater may cause damage.
- Do not leave InBodyBAND3 in a sauna or hot water.
- Do not leave InBodyBAND3 unattended or near high pressure water streams like waterfalls, waves, or underwater.
- Do not use InBodyBAND3 while playing extreme water sports such as diving, surfing, rafting, etc.
- If your product gets soaked with pure water, wipe it down with a soft, clean cloth until it is completely dry. If the liquid is not pure water (such as salt water, pool water, soapy water, oil, perfume, sunscreen, hand sanitizer, cosmetics, etc.), rinse the product with clean water first and then wipe it down with a soft, clean cloth until it is completely dry. Failure to follow this process may cause problems regarding the performance or appearance of the product.
- The optimal temperature for using InBodyBAND3 is 32 ~ 104 °F (0 ~ 40 °C), and the optimal temperature for storage is -4 ~ 140 °F (-20 ~ 60 °C). Temperatures outside these ranges may damage InBodyBAND3 or shorten the battery life.
- Do not cover or wrap InBodyBAND3 with a blanket while it is being used or charged. It may cause a fire.
- Do not keep InBodyBAND3 in a hot, humid environment (e.g., car, electric heating pad, etc.). InBodyBAND3 may be deformed, damaged, or explode.
- The battery life can be affected by different settings, amount of InBody Test, and InBody App usage.
- Do not touch the eyes directly with the running optical heart rate sensor. In particular, be careful not to put the measuring sensor to the eyes of infants. It may cause temporary or permanent eye injury.
- Do not disassemble or modify InBodyBAND3. It may cause product failure, measurement error, or personal injury. InBody USA will not be liable for any damage or loss caused by disassembling InBodyBAND3.
- Use InBodyBAND3 under the operating environment specified in InBodyBAND3 specifications.
- Only representatives from InBody service center are allowed to repair and inspect InBodyBAND3. Please contact the Product Support team at Support@inbody.com.

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E-mail: info.us@inbody.com

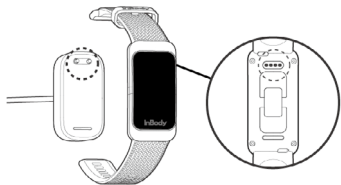
Copyright

Reproduction, adaptation, or translation of this manual is prohibited without prior written consent from InBody Co., Ltd. under the copyright laws. This manual might have typographical errors, and its content can be changed without prior notice. InBody Co., Ltd. shall not be liable for any errors, incidental, or consequential damages that occurred by not complying with the content of the User's Manual. InBody Co., Ltd. reserves the right to modify the appearance, specifications, and etc. of the InBody Dial to improve the quality of the product, without prior notice for reasons of performance improvement.

You can view and download the detailed guide on the InBody App and at <https://inbody.com>

1. Power On and Charging


Connect the charger to the InBodyBAND3 charging terminal.

- * It takes approximately 1 hour and 30 minutes to fully charge the discharged battery.
- * While the battery is being charged, only the current time and remaining battery life are shown.

2. Connecting the InBody App

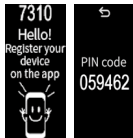- 1) Search InBody from Apple App store or Google Play store and install.
 - 2) Open the InBody App and register.
 - 3) Select InBodyBAND3 from the InBody App > ⚙ icon > Register (under Device) > InBodyBAND3
 - 4) Match the 4-digit serial number and PIN code shown at the top of the product screen with the InBody App.
- * When setup is complete, a time screen appears.

- * The InBodyBAND3 has to be registered on the InBody App to use.
- * Please update to the latest firmware version after the initial connection. The firmware can be updated from the InBody App > ⚙ icon > InBodyBAND3
- * InBody App provides InBody Test, step, sleep, heart rate, and stress results in more detail, and enables EZTraining.
- * If InBodyBAND3 is not connecting with the InBody App, try deleting InBodyBAND3 from your Bluetooth setting and reconnect it in InBody App.
- * Android OS 7.0 or later or iOS 13.0 or later.
- * Only smartphone models with Bluetooth version 5.0 or higher can be linked to InBodyBAND3.

3. Turning on the Display & Navigating

- 1) To turn on the display of the InBodyBAND3, rotate your wrist inward.
- 2) If you touch the screen, the display will turn on.
- 3) You can swipe up or down on the display to navigate.

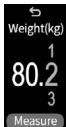
- * This product does not support the left-right scroll function.
- * If your fingers are dry, you may not be able to switch the screen smoothly.
- * You can change the type and sequence of the screen from the InBody App >  icon > InBodyBAND3 > Screen Setting > Display Setting.
- * Tap and hold the center of the screen for 3 seconds to move to the main screen.

4. InBody Test (Body Composition Analysis)

Go to the InBody Test screen and tap the center of the screen. Input your current weight. Then, tap "Measure" on the screen and cover the electrodes with your thumb and index finger.

InBody Test Screen

Weight Check / Test Start Screen

InBody Test Description Screen

InBody Test in Progress

Percent Body Fat / Skeletal Muscle Mass
Height, Weight / BMI

Make sure your height is entered correctly in the  icon (System) > Profile prior to testing.

Posture for Accurate InBody test

Put your fingers on the electrodes to touch with as much skin contact as possible.

Lightly press on the electrodes with two fingers. Make sure the InBodyBAND3 is stable on your wrist.

Be careful not to let the hands touch each other.

Both upper arms should not touch the body.

* You can view the test results on your InBody App.

5. Caution Before Measurement

Warning

- A person wearing an implanted medical device (e.g., pacemaker, etc.) or a life-sustaining device (e.g., patient monitoring device, etc.) should not use the InBodyBAND3. For example, the pacemaker may malfunction due to the current flowing through the body during InBody Test and this could pose a serious risk to your health and safety. InBody Co., Ltd. will not be liable for any injury or damage from violating these warnings.
- Bioelectrical Impedance Analysis (BIA) method uses micro-current and does no harm to the human body. However, if a pregnant woman uses InBodyBAND3, her fetus may be affected. Therefore, medical advice is required for such usage.
- Do not use the InBodyBAND3 in situations that are distracting and lead to dangerous circumstances or while driving a car.
- Since InBodyBAND3 is not a medical device, it is not intended to diagnose or give medical direction based on the symptoms of a disease.

Caution

- InBody Test results may be affected by environmental factors and the customer's measurement condition.
- InBodyBAND3 has a built-in magnet. Keep a safe distance from objects that can be affected by magnets, such as credit cards and medical devices.
- InBodyBAND3 was tested in a controlled environment and will be waterproof or dustproof only under the specified circumstance (IP68 requirements of IEC 60529 satisfied). Submerging the product underwater may cause damage.
- Do not leave InBodyBAND3 in a sauna or hot water.
- Do not leave InBodyBAND3 unattended or near high pressure water streams like waterfalls, waves, or underwater.
- Do not use InBodyBAND3 while playing extreme water sports such as diving, surfing, rafting, etc.
- If your product gets soaked with pure water, wipe it down with a soft, clean cloth until it is completely dry. If the liquid is not pure water (such as salt water, pool water, soapy water, oil, perfume, sunscreen, hand sanitizer, cosmetics, etc.), rinse the product with clean water first and then wipe it down with a soft, clean cloth until it is completely dry. Failure to follow this process may cause problems regarding the performance or appearance of the product.
- The optimal temperature for using InBodyBAND3 is 0 ~ 40 °C(32 ~ 104 °F), and the optimal temperature for storage is -20 ~ 60 °C(-4 ~ 140 °F). Temperatures outside these ranges may damage InBodyBAND3 or shorten the battery life.
- Do not cover or wrap InBodyBAND3 with a blanket while it is being used or charged. It may cause a fire.
- Do not keep InBodyBAND3 in a hot, humid environment (e.g., car, electric heating pad, etc.). InBodyBAND3 may be deformed, damaged, or explode.
- The battery life can be affected by different settings, amount of InBody Test, and InBody App usage.
- Do not touch the eyes directly with the running optical heart rate sensor. In particular, be careful not to put the measuring sensor to the eyes of infants. It may cause temporary or permanent eye injury.
- Do not disassemble or modify InBodyBAND3. It may cause product failure, measurement error, or personal injury. InBody USA will not be liable for any damage or loss caused by disassembling InBodyBAND3.
- Use InBodyBAND3 under the operating environment specified in InBodyBAND3 specifications.
- Only representatives from InBody service center are allowed to repair and inspect InBodyBAND3. Please contact the Product Support team at Support@inbody.com.

Contact

InBody Co., Ltd. [HQ]

InBody Bldg., 625, Eonju-ro, Gangnam-gu, Seoul 06106 KOREA

TEL: +82-2-501-3939 FAX: +82-2-6919-2417

Website: inbody.com E-mail: info@inbody.com

InBody Europe B.V. [NETHERLANDS]

Gyroscoopweg 122, 1042 AZ, Amsterdam, The Netherlands

TEL : +31-20-238-6080 FAX : +31-6-5734-1858

Website: nl.inbody.com E-mail: info.eu@inbody.com

Website (CS): csinbody.eu E-mail (CS): support@inbody.com

Copyright

Reproduction, adaptation, or translation of this manual is prohibited without prior written consent from InBody Co., Ltd. under the copyright laws. This manual might have typographical errors, and its content can be changed without prior notice. InBody Co., Ltd. shall not be liable for any errors, incidental, or consequential damages that occurred by not complying with the content of the User's Manual. InBody Co., Ltd. reserves the right to modify the appearance, specifications, and etc. of the InBody Dial to improve the quality of the product, without prior notice for reasons of performance improvement.

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- * "QR Code" is registered trademark of DENSO WAVE INCORPORATED.