LookinBody120

Data Management Software

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Installation Guide

Installation Guide **USA**

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LookinBody120

Data Management Software

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Installation Guide

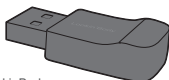
⚠ Check prior to installation

Do not plug in the Hardlock Key into the computer's USB port until instructed to do so by the installation wizard.

Hardlock Key

or

LookinBody
Bluetooth Dongle
(InBT400-USB or higher)

* If your InBody model supports Bluetooth connections, the supplied Bluetooth dongle will also act as the Hardlock Key.

Installation Steps for LookinBody120

Follow the steps below to set up the program:

1. Obtain the Installation File
Download the installation file provided by your supplier or vendor.
2. Follow the Installation Instructions
Carefully follow the detailed installation instructions for LookinBody120 to ensure proper setup.

Reinstalling LookinBody120

If you are reformatting or replacing your computer, follow these steps to reinstall LookinBody120.

1. Launch LookinBody120
Open the LookinBody120 application on your computer.
2. Access the Reinstallation Guide
Navigate to: [Setup] → [LookinBody Data Management] → [03. Reinstallation Guide]
3. Follow the Reinstallation Instructions
Complete the process by carefully following the on-screen instructions provided in the Reinstallation Guide.

System Requirements

Operating System: Windows 10 (32 bit/64 bit)

Processor: 1.8 GHz or faster processor recommended

Hard Disk: 10 GB or greater HDD free space

Memory: 4 GB RAM or greater

Resolution: 1280 X 1024 or higher

Input Device: Keyboard, Mouse, Trackpad

Connection Method: Serial (RS-232C), USB, LAN, Wi-Fi,
Bluetooth

* InBody Co., Ltd. reserves the right to modify the appearance, specifications, and etc. of LookinBody120 to improve the quality of the product, without prior notice for reasons of performance improvement.

**For any questions regarding
LookinBody120 or general questions,
contact:**

InBody Co., Ltd. [HQ]

InBody Bldg., 625, Eonju-ro, Gangnam-gu,
Seoul 06106 KOREA

TEL: +82-2-501-3939 FAX : +82-2-6919-2417

Customer Center: 1899-5841

Website: inbody.com

E-mail: info@inbody.com

InBody

LookinBody120

Data Management Software

Installation Guide **USA**

Installation Guide

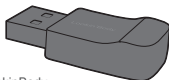
Check prior to installation

Do not plug in the Hardlock Key into the computer's USB port until instructed to do so by the installation wizard.

Hardlock Key

or

LookinBody
Bluetooth Dongle
(InBT400-USB or higher)

* If your InBody model supports Bluetooth connections, the supplied Bluetooth dongle will also act as the Hardlock Key.

How to Install

To install LookinBody 120, please use the download link provided and follow the on-screen instructions.

* Download link: <https://inbody.files.com/f/7d50b46beabe5afb>

If you are unable to install via the link, we can provide a USB with the installation files.

This is separate from the USB that contains the software license and Bluetooth function.

Reinstalling LookinBody120

If you are reformatting or replacing your computer, follow these steps to reinstall LookinBody120.

1. Launch LookinBody120
Open the LookinBody120 application on your computer.
2. Access the Reinstallation Guide
Navigate to: [Setup] → [LookinBody Data Management] → [03. Reinstallation Guide]
3. Follow the Reinstallation Instructions
Complete the process by carefully following the on-screen instructions provided in the Reinstallation Guide.

System Requirements

Operating System: Windows 10 (32 bit/64 bit)

Processor: 1.8 GHz or faster processor recommended

Hard Disk: 10 GB or greater HDD free space

Memory: 4 GB RAM or greater

Resolution: 1280 X 1024 or higher

Input Device: Keyboard, Mouse, Trackpad

Connection Method: Serial (RS-232C), USB2.0, LAN, Wi-Fi,
Bluetooth

* InBody Co., Ltd. reserves the right to modify the appearance, specifications, and etc. of LookinBody120 to improve the quality of the product, without prior notice for reasons of performance improvement.

**For any questions regarding
LookinBody 120,
contact:**

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©2025 InBody Co., Ltd. All rights reserved. IM-I1-D-25xxxx