InBody

DIAL H40

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Quick Start Guide

Quick Start Guide **USA**

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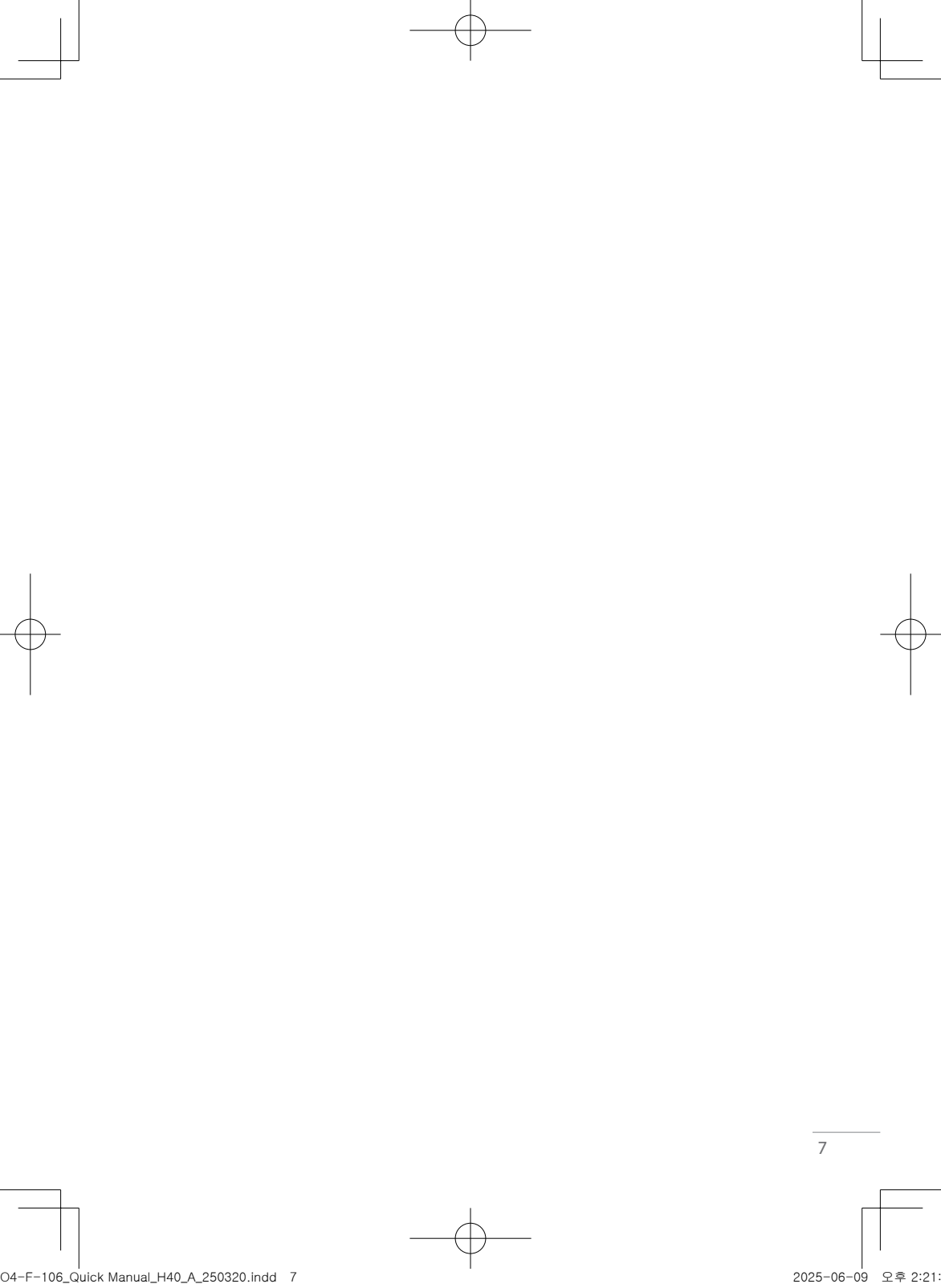
A detailed explanation can be checked on inbody.com or in the InBody App after registering the device in the InBody App.

1. Download the InBody App

- ① Search for InBody in the Apple App Store or Google Play Store to install the app.
- ② Open the InBody App and register the InBodyDial H40 in Settings > Register.
 - This can be connected to a mobile phone with iOS 13.0 or higher, Android 7.0 or higher, and Bluetooth 5.0 or higher.
 - To connect the InBody App with the device, the Bluetooth function on the mobile phone must be turned on.
 - When using the InBody App, you can record and manage your InBody Test result.

[Android User]

[iPhone User]

2. InBody Test Method

- ① Start the Test in the InBody App
 - 1) To start the test, select InBody App > Dashboard > Start the InBody Test.
 - To take the test, the InBodyDial H40 must be registered in the InBody App.
 - The Bluetooth function of the mobile device should be turned on, and for Bluetooth communication, the mobile device and Dial H40 should be kept close.
 - 2) Follow the guidance of the InBody App to complete the weight measurement and assume the test posture.
- ② Start the Test Using a Nickname Without Connecting to the InBody App
 - *In the InBody App, InBodyDial H40 and nickname registration must be completed.
 - 1) Step barefoot onto the footplate and turn on the power.
 - If the weight displayed is abnormal, step off the device and check for any debris on the floor. Make sure 0.0 kg is displayed, then measure your weight again.
 - 2) Once the weight measurement is complete, grab the handle and assume the test posture.
 - 3) Please check the nickname displayed on the screen.
 - Based on the analysis of existing data, the device displays the nickname associated with the recognized user.
 - You can choose another nickname by turning the dial.
 - If the height is changed, you can revise it in the InBody App.
 - The progress bar displays the remaining time (approximately 8 seconds) until the test starts.
 - For users who have not registered the device in the InBody App, turn the dial to select GUEST and input your height.
 - 4) Grab the handles and assume the test posture as shown in the 'Test Posture' picture.

3. Checking the InBody Test Results

① Check the Test Result on the Dial

1) After the test is completed, you can turn the dial to view the measurement results in order.

*When testing as User Recognition after starting the test or registering a nickname in the InBody App, you can also view the normal range graph.

② Check the test result in InBody App

1) You can check the test result in InBody App > Dashboard or InBody App > Detail.

2) If you complete the test using a nickname without an InBody App connection, you can check the test result saved in Dial H40 in InBody App > Sync Test Result.

- The Bluetooth function of the mobile device should be turned on, and for Bluetooth communication, the mobile device and Dial H40 should be kept close.

3) In the case of a Wi-Fi model, after completing the test with user recognition, the data will be transmitted to the InBody App. To save the test result, open the InBody App, select Dashboard, and press the Synchronization Start Button.

- Wi-Fi connection is available only on the Wi-Fi-supported model (H40NW).

- Wi-Fi must be registered and turned on in InBody App > Settings > InBody Dial H40.

- After completing the test and successfully transmitting the results, an app notification will be sent to the phone registered with the corresponding InBody account nickname.

Test Posture

1) Correct Arm Posture

- Your arms must not touch the sides of your body. If your arms touch the sides of your body, take the test with light clothing that has sleeves.

- Please lift the handle to the same height as your pelvis and do not place your arms on your stomach or lift them too high.

- Do not bend your arms.

2) Correct Hand Posture

- Grab the handle and place your thumb on the electrode.

- The left hand should not touch the right hand.

- Grab the handle to make sure that your hand and the electrodes stay in proper contact.

3) Correct Leg Posture

- Your thighs should not touch each other. If your thighs touch, please wear shorts or pants for the measurement.

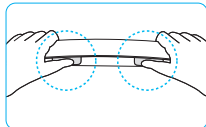
4) Correct Foot Posture

- Step barefoot, aligning your feet in the center of the electrodes.


- Do not wear socks or stockings when measuring.

- Your feet should not touch each other.


- When wearing long pants, make sure that no fabric gets caught between your foot and the foot electrodes.

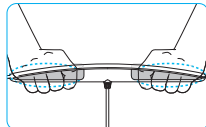
Please place your thumb on the rectangular electrodes.

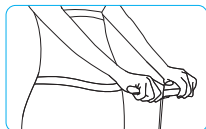
Place the inside of your four fingers on the backside electrodes of the handle.

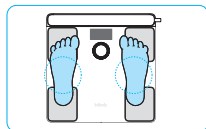Hold the handle at the height of your pelvis as shown in the picture, and stretch your arms to prevent them from touching your waist.

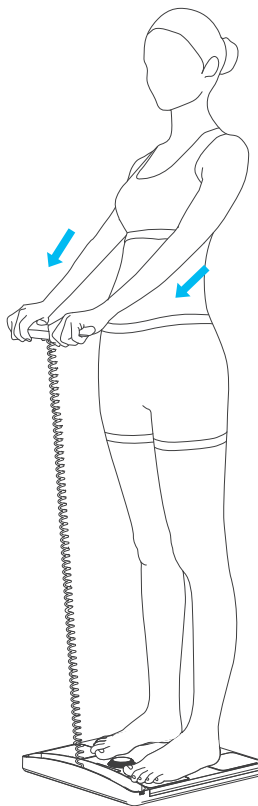Please position your feet in the center of the electrodes.

4. Precautions for Use

- Individuals with implanted medical devices, such as pacemakers or other medical devices essential for life support, must not use this device. For example, the pacemaker may malfunction due to the current flowing during the InBody Test, which could pose a risk to life. When the above precautions are not followed and damage to a person or the device occurs, InBody Co., Ltd. will not be held legally responsible.
- Bioelectrical Impedance Analysis (BIA) uses a weak electric current, so it does not cause any harm to the human body. If a pregnant woman uses this device, please consult with the appropriate physician or a medical expert.
- This device cannot diagnose diseases. To assess the presence of any diseases, consult with a doctor.
- Anyone unable to maintain the proper posture on their own should take the measurement with the help of guidance. For children, please assist them in maintaining the correct measurement posture.
- When storing the device for the long term, turn off the power, remove the batteries, and pack it. After that, store it on a flat surface.
- When disposing of the battery included with the device or the device itself, please follow the relevant regulations.
- In case of a device you purchased in Republic of Korea, InBody App connection is not supported that is logged in with global country number so you should log in in the Republic of Korea to connect with InBody App.

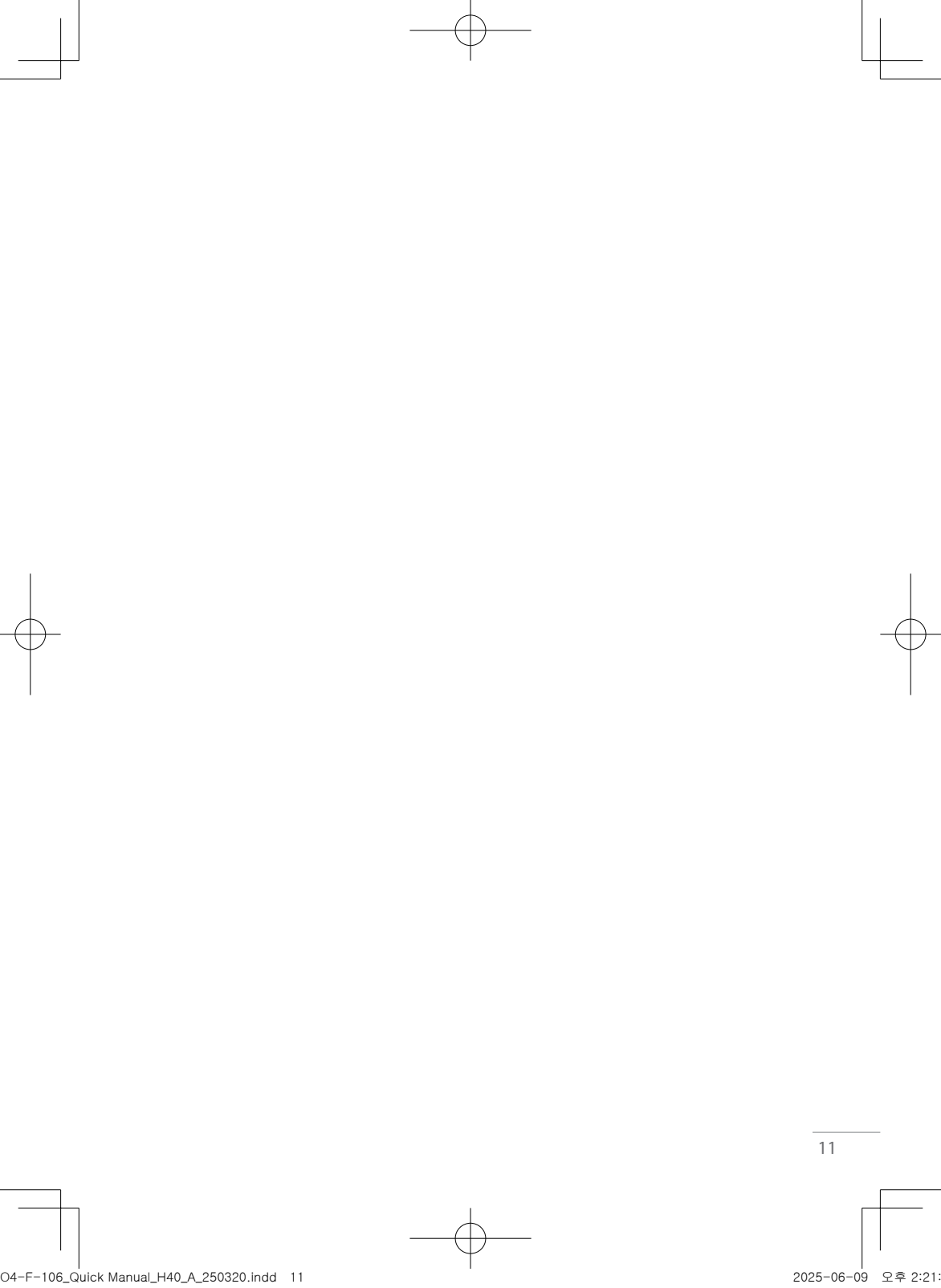
5. Device Warranty

- You can check the warranty policy on inbody.com or in the InBody App under 'More > Help > User Manual' after registering your device.
- Free warranty service will be provided within the warranty period only for a manufacturer's defect or a spontaneous failure.
- In the Republic of Korea, any matters not specified in this document will follow the consumer dispute resolution guidelines set by the Korea Fair Trade Commission. However, for users in other countries, the applicable local regulations will apply.
- In case of malfunctions or damages, please contact our customer service.

Copyrights

Reproduction, adaptation, or translation of this manual is prohibited without prior written consent from InBody Co., Ltd. under the copyright laws. This User's Manual may contain printing errors and is subject to change without prior notice. InBody Co., Ltd. shall not be liable for any errors, incidental, or consequential damages that occurred by not complying with the content of the User's Manual.

The appearance and specifications of the device may be changed without prior notice for performance improvements.

You can view and download the detailed guide in the InBody App after registering the device, or visit <https://inbodyusa.com>.

1. Download the InBody App

- ① Search for InBody in the Apple App Store or Google Play Store to install the app.
- ② Open the InBody App and register the InBodyDial H40 in Settings > Register.
 - Only mobile phones with iOS 13.0 or higher, Android 7.0 or higher, and Bluetooth 5.0 or higher can connect with the InBodyDial H40.
 - Enable the Bluetooth function on your mobile device to establish a connection with the InBody App.
 - You can record and manage your InBody Test results using the InBody App.

[Android User]

[iPhone User]

2. InBody testing method

- ① Start the examination in the InBody App
 - 1) To start the test, select InBody App > Dashboard > Start the InBody Test.
 - To take the test, the InBodyDial H40 must be registered in the InBody App.
 - The Bluetooth function on the mobile device must be turned on, and the mobile device should be kept close to InBodyDial H40 for proper Bluetooth communication.
 - 2) Follow the prompts of the InBody App to complete the weight measurement and assume the test posture.
- ② Start the Test Using a Nickname Without Connecting to the InBody App
 - *In the InBody App, InBodyDial H40 and nickname registration must be completed.
 - 1) Step barefoot onto the footplate and turn on the power:
 - If the weight displayed is abnormal, step off the device and check for any debris on the floor. Make sure 0.0 lb (0.0 kg) is displayed, then measure your weight again.
 - 2) Once the weight measurement is complete, grab the handle and assume the test posture.
 - 3) Please check the nickname displayed on the screen.
 - Existing data is analyzed to display a nickname for the user.
 - You can choose another nickname by turning the dial.
 - If the height is changed, you can modify it on the InBody App.
 - The progress bar displays the remaining time (approximately 8 seconds) until the test starts.
 - For users who haven't registered the device in the InBody App, turn the dial to select GUEST and input your height.
 - 4) Grab the handle and assume the test posture as shown in the 'Test Posture' picture.

3. Check InBody Test Results

① Check the Test Results on the Dial

1) After the test is completed, you can turn the dial to view the measurement results in order.

*When testing as user recognition after starting the test or registering a nickname in the InBody App, you can also view the normal range graph.

② Check the Test Results in the InBody App

1) You can check the test results in InBody App > Dashboard or InBody App > Detail.

2) If you complete the test using a nickname without an InBody App connection, you can check the test result saved in the InBodyDial H40 in InBody App > Sync Test Result.

- The Bluetooth function on the mobile device must be turned on, and the mobile device should be kept close to InBodyDial H40 for proper Bluetooth communication.

3) In the case of a Wi-Fi model, after completing the test with user recognition, the data will be transmitted to the InBody App. To save the test result, open the InBody App, select Dashboard, and press the Synchronization Start Button.

- Wi-Fi connection is available only on the Wi-Fi-supported model (H40NWi).

- Wi-Fi must be registered and turned on in InBody App > Settings > InBodyDial H40.

- After completing the test and successfully transmitting the results, an app notification will be sent to the phone registered with the corresponding InBody account nickname.

Test Posture

1) Correct Arm Posture

- Your arms must not touch the sides of your body. If your arms touch the sides of your body, wear light clothing with sleeves when taking the test.
- Please lift the handle to the same height as your pelvis and do not place your arms on your stomach or lift them too high.
- Do not bend your arms.

2) Correct Hand Posture

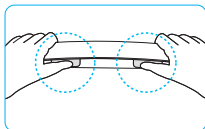
- Grab the handle and place your thumbs on the electrodes.
- The left hand should not touch the right hand.
- Grab the handle to make sure that your hand and the electrodes stay in proper contact.

3) Correct Leg Posture


- Your thighs should not touch each other. If your thighs touch, please wear shorts or pants for the test.

4) Correct Foot Posture


- Stand barefoot on the center of the electrodes.
- Do not wear socks or stockings when taking the test.
- Your feet should not touch each other.
- When wearing long pants, make sure that no fabric gets caught between your feet and the foot electrodes.

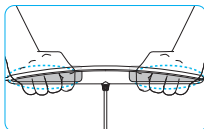
Please place your thumb on the rectangular electrodes.

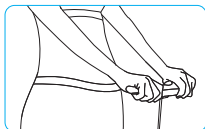
Place the inside of your four fingers on the backside electrodes of the handle.

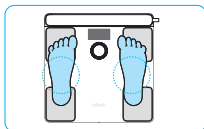Hold the handle at the height of your pelvis as shown in the picture, and stretch your arms to prevent them from touching your waist.

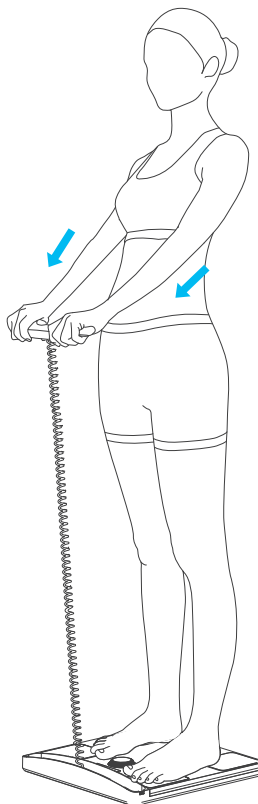Please position your feet in the center of the electrodes.

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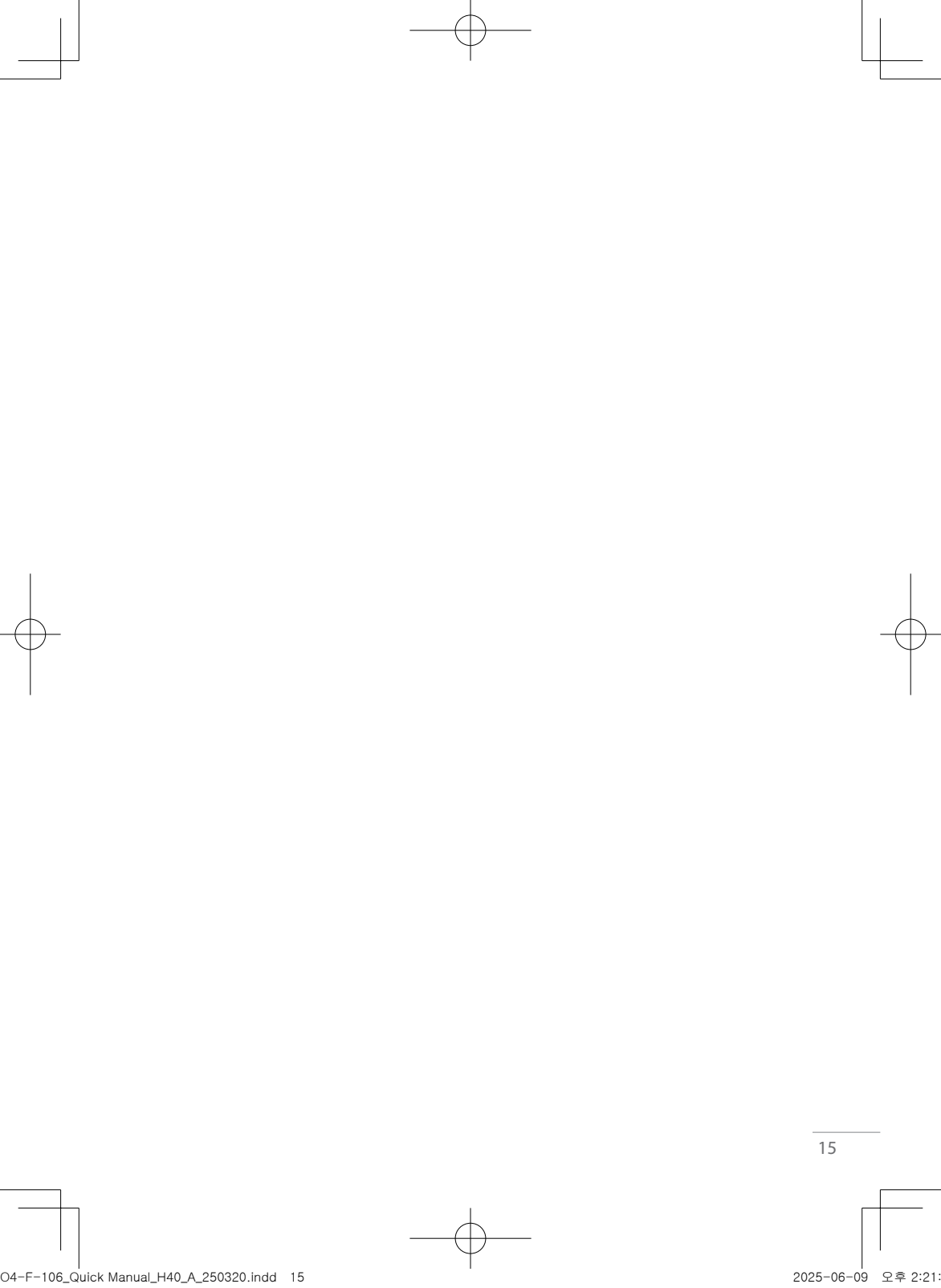
5. Device Warranty

- You can check the warranty policy on inbody.com or in the InBody App under 'Settings > Help > User Manual' after registering your device.
- Warranty service will be effective only within the warranty period provided and only for an inherent manufacturer's defect.
- In the Republic of Korea, any matters not specified in this document will follow the consumer dispute resolution guidelines set by the Korea Fair Trade Commission. However, for users in other countries, the applicable local regulations will apply.
- In case of malfunctions or damages, please contact our customer service.

Copyrights

Reproduction, adaptation, or translation of this manual is prohibited without prior written consent from InBody Co., Ltd. under the copyright laws. This User's Manual may contain printing errors and is subject to change without prior notice. InBody Co., Ltd. shall not be liable for any errors, incidental, or consequential damages that occur due to non-compliance with the content of the User's Manual.

The appearance and specifications of the device may be changed without prior notice for performance improvements.

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