

Compatible Printer List

Not all printer options are compatible for each InBody model. Before purchasing a new printer, check the INBODY PRINTER SETUP INSTRUCTIONS on page two, to confirm your device has the needed printer language setting to accommodate your desired printer. Please contact support@inbody.com if you have questions or need further assistance.

Manufacturer	Printer Model	Type	Printer Language	Interface
HP	LaserJet Pro M102w	Laser	PCLmS	USB
HP	LaserJet Pro M15w	Laser	PCLmS	USB
HP	LaserJet Pro M29w	Laser	PCLmS	USB
HP	LaserJet M110w	Laser	PCLmS	USB
HP	LaserJet Pro M209dw	Laser	PCLmS	USB
HP	Neverstop Laser 1001nw	Laser	PCLmS	USB
HP	LaserJet Pro M203dw	Laser	PCL	USB
HP	ENVY 6075	InkJet	PCL 3 GUI	USB
HP	ENVY Pro 6475	InkJet	PCL 3 GUI	USB
Brother	HL-L2370D	Laser	PCL	USB
Brother	HL-L2380D	Laser	PCL	USB
Brother	HL-L3230CDW	Laser	PCL	USB
Brother	HL-L3210CW	Laser	PCL	USB
Brother	HL-5100DN	Laser	PCL	USB
Pantum	P3300 Series	Laser	PCL	USB

InBody Printer Setup Instructions



1. SELECT "Administrative Menu" and INPUT your password (0000 is the default code unless it has been changed by your facility)
2. SELECT "09. Printer Setup"
3. ENSURE "USB cable" and "O" are selected before SELECTING "Next"
4. Follow steps 1-3 to connect your printer with a USB cable and SELECT "Next" once completed. (Note: USB cable connection can be bypassed at this time if you are only searching for your printer languages)
5. SELECT "O" to proceed to your InBody available language options
- If you do not wish to currently setup a printer, SELECT the "Exit" button until you return to the main screen. If you wish to continue setup of your printer, continue to step 5
6. Search and SELECT your compatible printer language based on the Compatible Printer List and SELECT "Test Print"
7. Upon successful print, press "O" to complete your printer setup

